Immediate Medical Support You Can Rely On.

Philadelphia[™] Cervical Collar[®] Company rewards the Trauma and E.M. FIELD Professionals by putting better and more efficient quality tools in their hands to comfort their patients and satisfy themselves.

The E.M. Adult Collar

The ease of use of the Philadelphia[™] Cervical Collar[®] Company new red E.M. Collar's easy application is demonstrated in the following text and photographs.

Figure 1. Flatten back portion of collar. Place index finger in "looped" part of bow and hold edge of collar with thumb as you simultaneously slide bow and collar under the patient's neck. Just apply and center back portion of collar if patient is in sitting position.

Figure 2. Place front portion of collar making sure chin is in proper position. Pull with just enough pressure to secure the head.

Figure 3. If patient is nauseous, just pull chin piece down enough to allow vomit to escape to avoid an obstruction in the airway. If patient is strapped in spineboard, tilt board.

Several additional items of importance to note are:

Most collars do not support the arch of the cervical spine. The new red E.M. Philadelphia™ Cervical Collar® is one of the few collars in the E.M. field that has a preformed back that conforms to the spine to accommodate support to the cervical arch.

The new red E.M. Philadelphia™ Cervical Collar® is the one collar in the E.M. field that supports the head carefully without tilting it from side to side and it eliminates rotation because of the Head Immobilizer attachment.

The new red E.M. Philadelphia[™] Cervical Collar[®] offers three sizes that fit most adults and children. Patients with short necks will be accommodated soon.

All new red E.M. Philadelphia[™] Cervical Collar[®] products come with detailed instructions.


CAUTION: ALL Philadelphia™ Cervical Collar® Products should be applied by a physician or certified medical authority

Call or write for your local dealer:


Philadelphia Collar Company


P.O. Box 185 • Westville, NJ 08993 (215) 923-9760

Patents: U.S. 3,756,226; 4,677,969; 4,502,471; 4,515,153; Canadian (Brevete) 1976, 1986; U.K. 2,126,485; DBP 3534191C2

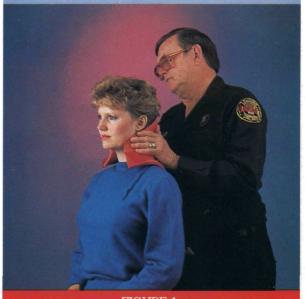



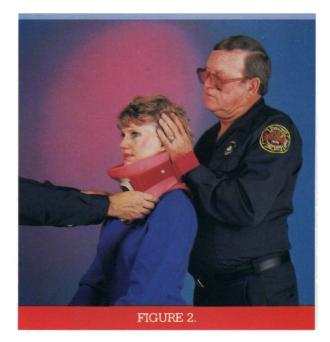

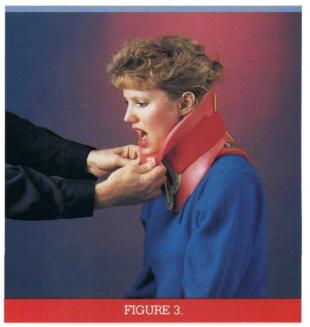

Laerdal COMPACT SUCTION UNIT

A totally NEW Emergency Aspirator



